



GIG
CYMRU
NHS
WALES

Bwrdd Iechyd Prifysgol
Betsi Cadwaladr
University Health Board

Airsense 10
'Triple 10 Therapy'
Quick Start Guide

TRAINED INDIVIDUALS ONLY! (see back page)

1 Fit CPAP tubing into back of machine



2



Fit filter to other end
Ensure there is a holy T-Piece
Make sure all ports/caps closed

3 Add correctly-sized mask




4



Add green oxygen tubing to mask
Ensure it fits snugly and won't fall out


5 Plug other end of green tube into oxygen flowmeter.
Turn flow to **10 Litres/Min**
DO NOT CHANGE THIS

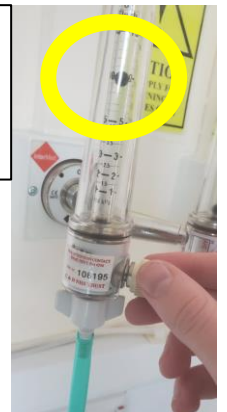

Turn Over


Triple 10 Therapy: Back Page

- 6** Plug in machine switch on at wall.
Wait for this screen to appear.

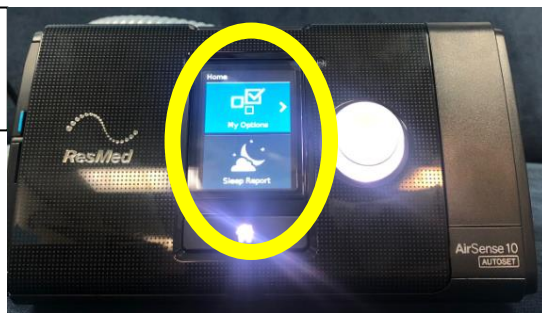

Now press the green button.
Flow will start.

- 8** You will see this screen:
Number will build up to 10

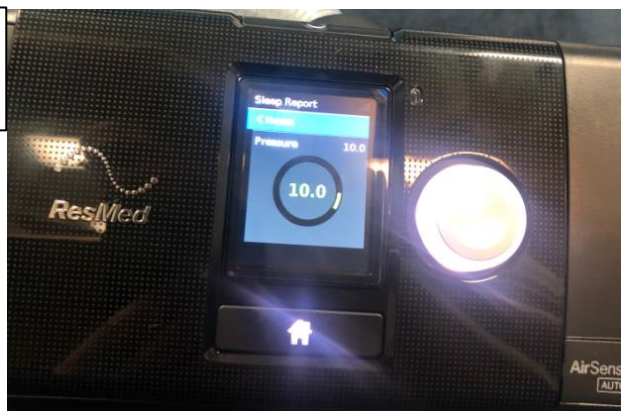

NB if the machine is not attached to a patient the flow will stop after about 10 seconds – just press the green button and it will start again.

DO NOT TOUCH THE CONTROLS

DO NOT CHANGE THE OXYGEN

This is 10cmCPAP and 10 litres O₂ on the AirSense 10 – you are now delivering Triple 10 Therapy!

To get a machine

0800-2000: Bleep 4290 to arrange for machine and disposables
Out of Hours: Bleep 4400
Comes with range of mask sizes
ONLY OPEN ONE PACKET
Ensure unopened packets are kept for collection

For more info/training

Contact ICU or Respiratory Nurses.
Training materials available on StaffConnect app
Or point your camera at the QR code!



POINT CAMERA AT ME